EN Lyna II + wheels

Additional to the manual Lyna II, Lindsey.

The walking frame Lyna II can be changed to a rollator by mounting four wheels under the walking frame.

1. Technical Specifications

Description	Dimensions
Total width	540 mm
Total length	570 mm - 600 mm
Maximum height	1000 mm
Minimum height	820 mm
Width (folded)	575 mm
Length (folded)	850 mm - 1020 mm
Height (folded)	120 mm
Distance between handgrips	495 mm
Weight	3,4 kg
Maximum inclination forwards	11,5°
Maximum inclination backwards	7,5°
Maximum inclination sideways	0°
Maximum user weight	130 kg
Turning diameter	1260 mm
Maximum angle handgrips	0° (not articulating) 3,5° (articulating)
Storage temperature	+5°C - +41°C

We reserve the right to introduce technical changes.
Measurement tolerance +/- 15mm / 1,5 kg / °

2. Check after delivery

Unpack your product and verify if the delivery is complete. Following items should be included:

- Walking frame Lyna II
- Wheels (4x)
- Manual

Verify your product for transport damage. If you find any damages after delivery proceed as follows:

- Contact the transporter
- Have a list made of any problems
- Contact your supplier


3. Safety rules

- ⚠ Check that the wheels / tips are properly mounted to the walking frame.
- ⚠ The walking frame is only for indoor use.


4. Mounting / Removing wheels

Mounting wheels:

1. Press on the locking pin with your thumb.
2. Move the lower leg downwards and remove these completely from the walking frame.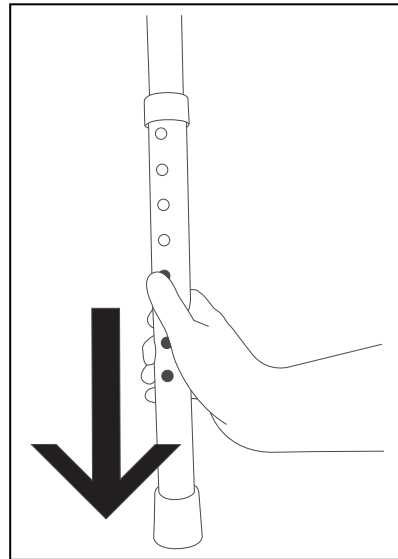

3. Grasp the wheel.
4. Mount the wheel to the walking frame and move the wheel upwards to the desired height.

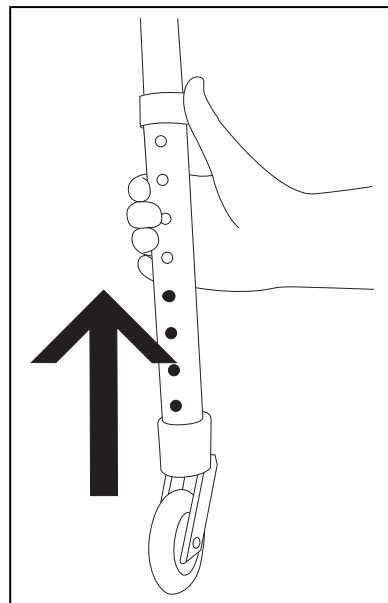5. Check that the locking pin is firmly clicked in one of the holes and the wheel is secured to the walking frame.
6. Repeat instructions above for the three other wheels.

For removing the wheels and mounting the tips you can use same instructions.

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