# VERMEIREN

## Elios

INSTRUCTION MANUAL  
MODE D'EMPLOI  
GEBRUIKSAANWIJZING  
GEBRAUCHSANWEISUNG  
ISTRUZIONI PER L'USO  
MANUAL DE INSTRUCCIONES  
INSTRUKCJA OBSŁUGI

## **Instructions to the specialist dealer**

This instruction manual is part and parcel of the product and must accompany every product sold.

Version: B, July 2012

All rights reserved, including translation.

No part of this manual may be reproduced in any form what so ever (print, photocopy, microfilm or any other process) without written permission of the publisher, or processed, duplicated or distributed by using electronic systems.

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© N.V. Vermeiren N.V. 2012

## Contents

<b>Preface</b> .....	<b>2</b>
<b>1 Productdescription</b> .....	<b>3</b>
1.1 Intended Use.....	3
1.2 Technical specifications .....	4
1.3 Drawing.....	5
1.4 Location identification plate.....	5
1.5 Explanation of symbols.....	5
1.6 Accessories.....	6
1.7 For your safety .....	6
<b>2 Use</b> .....	<b>6</b>
2.1 Carrying the shell chair .....	6
2.2 Recommendations to use the shell chair Elios.....	6
2.3 Inclination of the shell chair.....	8
2.4 Parking brakes.....	8
2.5 Use of the safety belt.....	9
2.6 Mounting or removing the table.....	10
2.7 Mounting or removing the foot bracket .....	10
2.8 Adjusting seat comfort .....	11
2.9 Taking curbs by using ramps.....	11
2.10 Transport in the car.....	11
<b>3 Installation and adjustment</b> .....	<b>11</b>
3.1 Tools.....	12
3.2 Manner of delivery .....	12
3.3 Adjusting the seat height.....	12
<b>4 Maintenance</b> .....	<b>13</b>
4.1 Regular Maintenance.....	13
4.2 Shipping and storage .....	13
4.3 Care .....	13
4.4 Inspection .....	14
4.5 Disinfection .....	14
<b>5 Guarantee</b> .....	<b>16</b>
<b>6 Disposal</b> .....	<b>16</b>
<b>7 Declaration of conformity</b> .....	<b>16</b>
<b>8 Maintenance plan</b> .....	<b>17</b>
<b>9 Disinfection book</b> .....	<b>17</b>

## Preface

First of all we want to thank you for putting your trust in us by selecting one of our products.

Vermeiren products are the result of many years of research and experience. During the development, special attention was given to the ease of use and the serviceability of the product.

The lifetime for the product is strongly influenced by the care and maintenance of the product.

This manual will help you get acquainted with the operation of your product.

Following the user instructions and the maintenance instructions are an essential part of the warranty.

This manual reflects the latest product developments. Vermeiren has the right to introduce changes without the obligation to adapt or replace previously delivered models.

For any further questions, please consult your specialist dealer.

# 1 Productdescription

## 1.1 *Intended Use*

The Elios shell chair is developed entirely for your comfort.

The shell chair is designed for seating 1 person.

The shell chair has been designed exclusively for people with limited mobility.

The shell chair is suited for indoor use.

You will need a helper for indoor transportation of the Elios.

The different types of accessories, and the construction allow full use by persons disabled by:

- paralysis
- weak/malformed limbs
- injuries of the joints
- heart insufficiencies and poor blood circulation
- balance disturbances
- cachexia (decrease in muscle)
- and also for aged persons.

The indication to use this product must take into account the following elements:

- body size and weight (**max. load 120 kg**)
- physical and psychological condition
- home environment

The shell chair should only be used on surfaces where all four wheels are touching the ground and where there is sufficient contact to propel the wheels equally.

The shell chair should not be used as a ladder, nor should it be used for transportation of heavy or hot objects.

It is necessary to choose the best fitting model for the size/build of the patient.

Use only Vermeiren approved accessories.

The manufacturer is not liable for damage caused by the lack of or improper service or as a result of not following instructions from this manual.

Compliance with the user and maintenance instructions are an essential part of the guarantee conditions.

## 1.2 Technical specifications

All dimensions are measured with upholstery.

<b>Make</b>	Vermeiren		
<b>Address</b>	Vermeirenplein 1/15, B-2920 Kalmthout		
<b>Type</b>	Shell chair		
<b>Model</b>	Elios		
<b>Maximum occupant mass</b>	120 kg		
<b>Description</b>	<b>Min.</b>		<b>Max.</b>
Overall length with leg rest	1480 mm		1620 mm
Effective seat width	400 mm	440 mm	480 mm
Overall width	690 mm		
Overall height	1320 mm		1385 mm
Total mass	Approx. 46 kg (with table)		
Mass of heaviest part	Seat 13 kg		
Maximum load of table	3 kg		
Static stability downhill	6% (in standard configuration)		
Static stability uphill	6% (in standard configuration)		
Obstacle climbing	Not applicable		
Seat plane angle	0°		+37°
Effective seat depth	510 mm		
Seat surface height at front edge	465 mm		530 mm
Backrest angle	0°		+37°
Backrest height	750 mm		790 mm
Length from seat to leg rest end	350 mm		450 mm
Angle between seat and leg rest	69,45°		180°
Armrests height	240 mm		
Color of upholstery	Taupe		
Storage and use temperature	+ 5 °C		+ 41 °C
Storage and use humidity	30%		70%
<b>We reserve the right to introduce technical changes. Measurement tolerance ± 15 mm / 1,5 kg / °</b>			


*Table 1: Technical specifications*

The shell chair complies to the requirements set up in:


ISO 7176-8: Requirements and test methods for static, impact and fatigue strengths.

EN 1021-1, EN1021-2: Resistance to ignition of upholstered parts

### 1.3 Drawing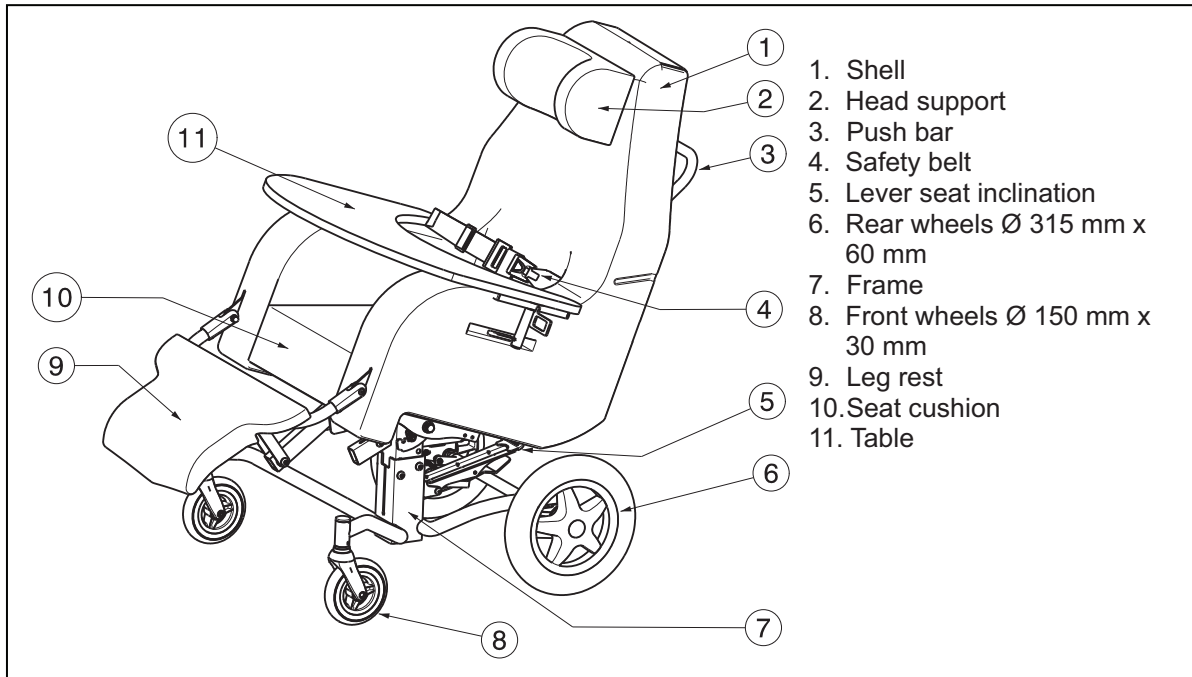

### 1.4 Location identification plate

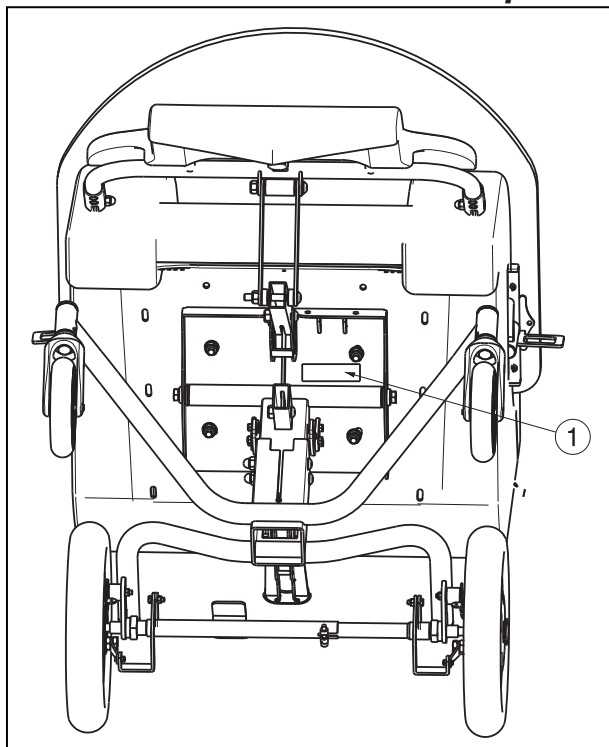The location of the identification plate ① is on the lower frame on the bottom side of the shell chair.

### 1.5 Explanation of symbols

- Maximum mass
- Indoor use
- CE conformity
- Not crash-tested

## 1.6 Accessories

**⚠ WARNING:** Precautions should be taken if accessories not listed below are used.

- Serum holder

## 1.7 For your safety

Take note of the following safety hints:

- ⚠ To prevent injury of the hands and damage to the shell chair, make sure that no objects and or body parts are caught between the adjustment systems (seat height, inclination of the shell chair, ...)
- ⚠ Regularly check that the wheels, brakes function correctly.
- ⚠ The parking brakes should be applied before getting into and out the shell chair.
- ⚠ If you want to pick up something (lying in front of, on the side, or to the rear of the shell chair), you should not lean too far out to avoid tipping over.
- ⚠ When moving through doors, arches, etc. ensure that there is enough room at the sides so there is no damage to the shell chair.
- ⚠ Only use your shell chair according to regulations. For example, avoid uncontrolled rolling against obstacles (steps, curbs, doorframes, etc) or dropping down from ledges. The manufacturer cannot assume liability for damage caused by overloading, collision or other improper use.
- ⚠ Never cross steps, ledges or other obstacles with a patient seated in the chair. If alternatives like ramps or lifts are available, please use them.
- ⚠ When transporting the shell chair, never pick it up by grasping movable parts (wheels).
- ⚠ Never exceed the **maximum load (120 kg)**.
- ⚠ Use only genuine and original parts for maintenance work.

## 2 Use

This chapter describes the everyday use. **These instructions are for the user and the specialist dealer.**

The product is delivered fully assembled by your specialist dealer. The instructions intended for the specialist dealer on how to set up the shell chair are given in § 3.

### 2.1 Carrying the shell chair

The best way to carry the shell chair is to make use of the wheels and roll the shell chair to the desired place, after releasing the brakes.

### 2.2 Recommendations to use the shell chair Elios

#### 2.2.1 Transfer in and out the chair

- ⚠ **WARNING:** Risk of injury – Adjust the safety belt as close as possible to the body of the patient so the patient cannot slide out of the safety belt.
- ⚠ **WARNING:** Risk of tipping over – Be sure that the shell chair does not tip over backwards when the shell chair is placed in his most backwards position.
- ⚠ **CAUTION:** In case you cannot perform the transfer in a safe manner, ask someone to assist you.

The transfer of the patient into the shell chair shall be according the instructions below:

1. Position the shell chair as close as possible to the chair, couch or bed from you wish to transfer.
2. Check the brakes from the shell chair are engaged and that the safety belt is firmly attached to the shell chair.
3. Place the shell chair in the most forward position (minimum inclination: 0°).
4. Transfer to the chair.
5. Fasten the safety belt.
6. Check that the safety belt is correctly mounted (as close as possible to the body of the patient). If needed adjust the safety belt.
7. Adjust the shell chair to a comfortable position (Inclination: 0° - 37°), if necessary.

The transfer of the patient out of the shell chair shall be according the instructions below:

1. Position the shell chair as close as possible to the chair, couch or bed to which you wish to transfer.
2. Check the brakes from the shell chair are engaged.
3. Place the shell chair in the most forward position (minimum inclination: 0°).
4. Unfasten the safety belt.
5. Transfer the patient out the chair.

### 2.2.2 Correct position in the chair

Some recommendations for a comfortable use of the chair:

- Position your backside as close as possible to the backrest.
- Make sure your upper legs are lying comfortably on the seat cushion– If needed adjust the length of the leg rest.

### 2.2.3 Moving/transporting the chair

**⚠ WARNING: Risk of clamping – Be careful passing through restricted passages (e.g. doors).**


**⚠ WARNING: Risk of burns – Be careful when using in hot or cold environments for a sufficient amount of time and when touching - Surfaces can assume the environment temperatures.**

**⚠ WARNING: Risk of injury – There should be no persons or objects under the shell chair during transportation.**

1. Engage the brakes.
2. Transfer the patient out the chair.
3. Disengage the brakes.
4. Transport the chair to the desired place on an horizontal underground.
5. Transfer the patient in the chair.
6. Engage the brakes.

### 2.3 Inclination of the shell chair

- ⚠ **WARNING:** Risk of tipping over – Be sure that the shell chair does not tip over backwards when the shell chair is placed in his most backwards position.
- ⚠ **WARNING:** Risk of injury – Check the brakes are engaged before changing the inclination of the shell chair.
- ⚠ **WARNING:** Risk of injury – Check that the shell chair is firmly locked after inclination.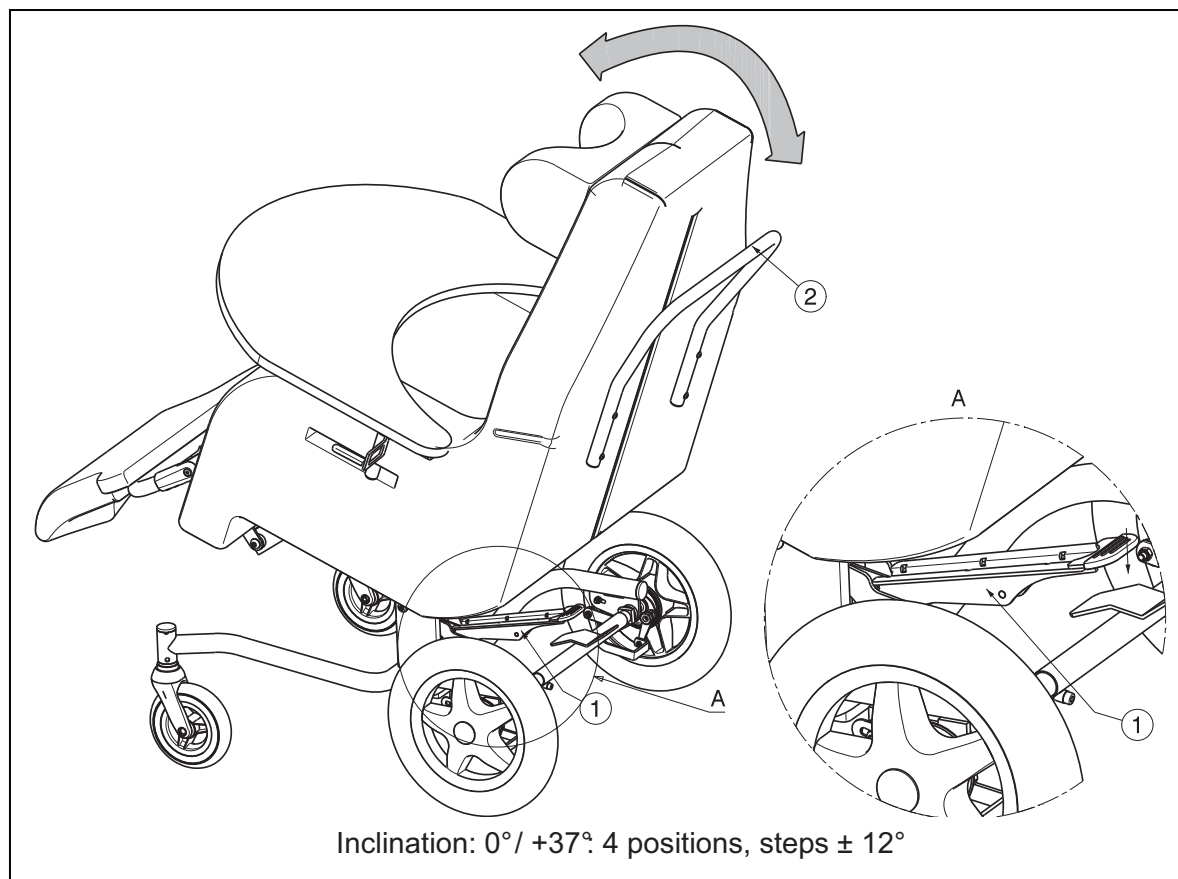


Inclination set by an attendant:

1. Engage the brakes.
2. Push the inclination lever ① downwards.
3. Pull/Push the push bar ② in the desired direction to incline the shell chair backwards or set in the horizontal position (0° to +37°).
4. The leg rest shall also be inclined.
5. Release the inclination lever ①.
6. Check that the shell chair is firmly locked, by pushing and pulling the shell chair with the push bar ② gently down and upwards.

### 2.4 Parking brakes

- ⚠ **WARNING:** Risk of injury – Engage the brakes to put the shell chair in a parking position.
- ⚠ **WARNING:** Good operation of the brakes is influenced by wear and contamination of the tyres (water, oil, mud, ...) – Check the condition of the tyres before each use.
- ⚠ **WARNING:** The brakes are adjustable and can wear – Check the operation of the brakes before each use.
- ⚠ **CAUTION:** Risk of unintended movement – Make sure the shell chair is on a flat horizontal surface before releasing the brakes.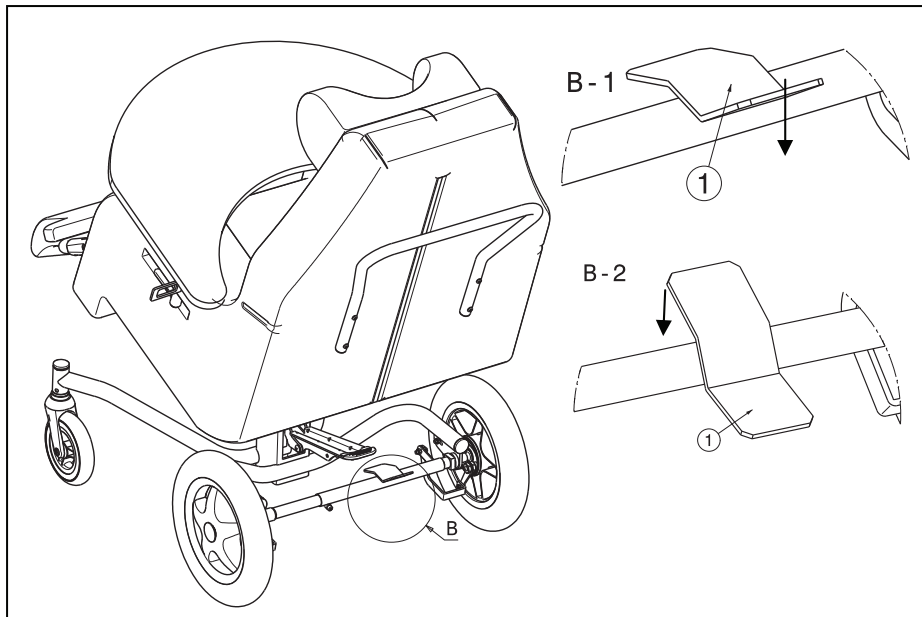

#### **Engaging the parking brakes (Picture B-1)**

The brakes are located on the back side of the shell chair. With your foot, press the back side of the brake plate ① downwards.


#### **Disengaging the parking brakes (Picture B-2)**

The brakes are located on the back side of the shell chair. With your foot, press the front side of the brake plate ① downwards.

## **2.5 Use of the safety belt**


- ⚠ **WARNING: Risk of injury – Adjust the safety belt as close as possible to the body of the patient so the patient cannot slide out of the safety belt.**
- ⚠ **WARNING: Risk of injury – Use the safety belt for patients with extreme movements. Failure to do this could lead to the patient falling out of the chair.**

When the patient has spasms or makes restless movements the safety belt shall be used.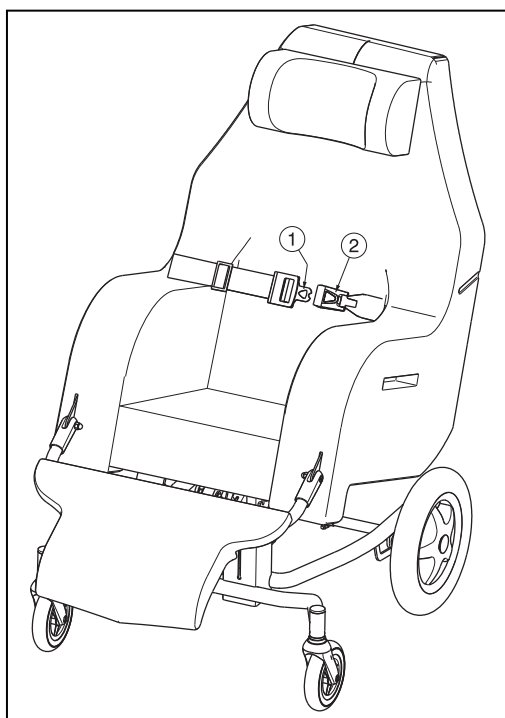

1. Mount the waist belt around the waist of the patient.
2. Click the buckle of belt ① into the buckle of belt ②.
3. If needed adjust the waist belt.

The adjustment of the belt shall be with the plastic adjustment system ① mounted on the belt.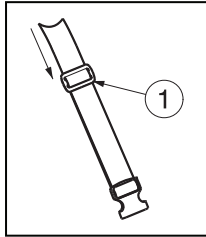


1. Pull the belt through the adjustment system ①.

## 2.6 Mounting or removing the table

**⚠ WARNING: Risk of injury – Do not exceed the maximum load of 3 kg.**

**⚠ WARNING: Risk of injury – Check that the table is attached correctly.**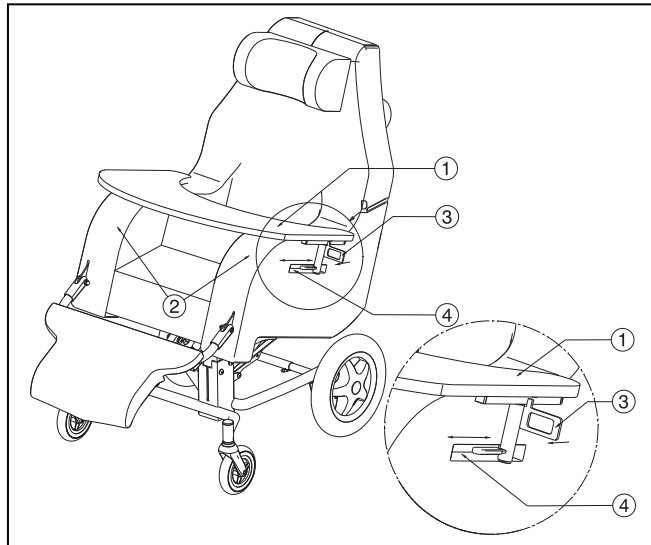

Mounting the table:

1. Place the table ① on the armrests ②.
2. Push the lock system ③ on both sides, in the grooves ④ of the shell chair, to attach the table.
3. Check that the table is attached correctly.

Removing the table:


1. Loosen the lock system ③ on both sides.
2. Remove the table ①.

Adjusting the table:

1. Loosen the lock system ③ on both sides.
2. Move the table in the grooves ④ until the desired position is reached.
3. Push the lock system ③ on both sides, in the grooves ④ of the shell chair, to attach the table.
4. Check that the table is attached correctly.

## 2.7 Mounting or removing the foot bracket

**⚠ WARNING: Risk of injury – Check that the foot bracket is attached correctly.**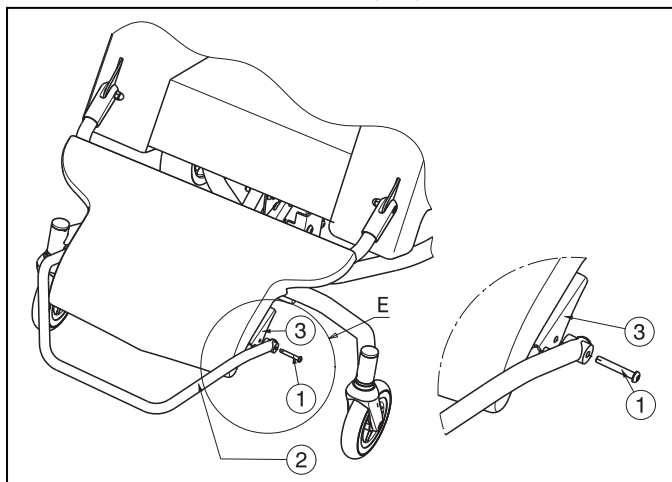

Mounting the foot bracket:


1. Hold the foot bracket ② against the plate ③.
2. Place the screw ① in the hole.
3. Check that the foot bracket is attached correctly.

Removing the foot bracket:

1. Remove the screw ①.
2. Remove the foot bracket ②.

## 2.8 Adjusting seat comfort

Use instructions below to adjust the seat comfort for the patient :

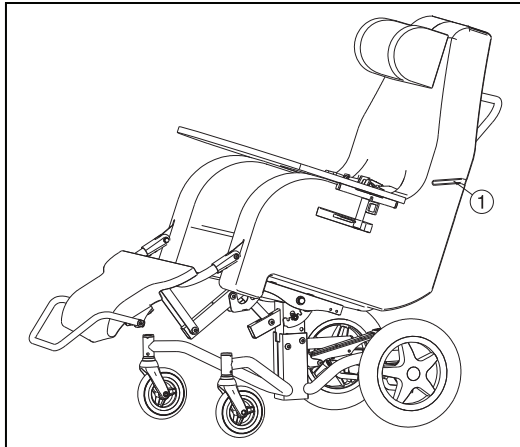1. Open the zipper ① on the side of the shell chair.
2. Take the big pump (air in) or small pump (air out) out of the pocket on the side of the shell chair.
3. Pump the upholstery to adjust the seat comfort. If you pump the air in the upholstery, be careful you do not pump too much air in the upholstery.

## 2.9 Taking curbs by using ramps

**⚠ WARNING: Risk of tipping over – Never go up or down ramps without the aid of a helper.**

If you are considering the installation of ramps to overcome obstacles, kindly note the following:

The chair is intended first and foremost to be used inside on level and solid surfaces. Never go up or down ramps that are rough or have bumpy edges. Limit yourself to ascents or descents with a maximum angle of 6%.

## 2.10 Transport in the car

**⚠ DANGER: Risk of injury – The shell chair is not suited for use as a seat in a motor vehicle.**

**⚠ WARNING: Risk of injury – See that the chair is attached properly in the car or the truck.**

Never use your shell chair as a seat in an automobile or other vehicle. The shell chair shall be marked with following symbol.

To transport the shell chair in the car use following steps:

1. Remove accessories.
2. Store accessories safely.
3. Place the chair in the luggage place.
4. If the chair and the passenger compartment is NOT separated, attach the frame of the chair securely to the vehicle. You can use the available safety belts in the vehicle.

## 3 Installation and adjustment

The instructions in this chapter are for the specialist dealer.

To find a service facility or specialist dealer near you, contact the nearest Vermeiren facility. A list of Vermeiren facilities can be found on the last page.

**⚠ WARNING: Risk of unsafe settings - Use only the settings described in this manual.**

**⚠ WARNING: Variation of allowed adjustments can still change the stability of your product (tilt back or sideways).**

### 3.1 Tools

To set up the chair the following tools are needed.

- Wrench set n° 10, 17
- Allen key set n° 4, 6

### 3.2 Manner of delivery

The Vermeiren product shall be delivered with:

- 1 frame with shell and 4 wheels
- 1 leg rest + foot bracket
- Tools
- Manual
- Accessories
- 2 Pumps

Before use check if everything is included and that no products are damaged (example by transport, ...).


### 3.3 Adjusting the seat height

- ⚠ **WARNING:** Risk of injury – Make sure that no objects and or body parts are caught between the adjustment systems.
- ⚠ **WARNING:** Risk of injury – Adjusting can only be done by authorized people.
- ⚠ **WARNING:** Risk of injury – Check that all screws have been properly fastened.

To adjust the seat height you can set the shell chair at different heights on the frame. The shell chair must not be removed during this process.

Seat height	Position
465 mm (Standard)	Holes 1
470 mm	Holes 2
490 mm	Holes 3

Table 2: Seat heights

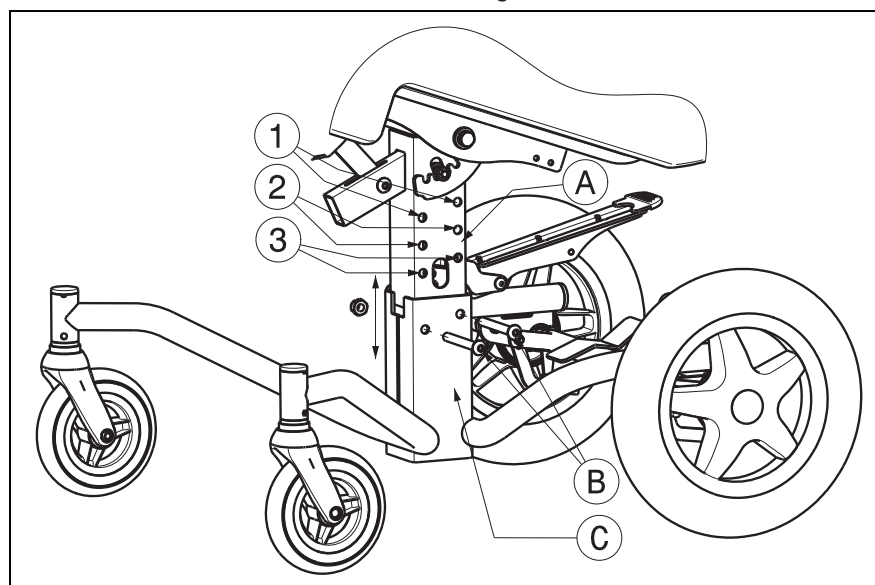1. Place the shell chair on his side.
2. Loosen the screws ②.
3. Move part ① out part ③.
4. Place the shell chair in the desired position (refer to table 2).
5. Retighten the screws ② firmly by hand.

## 4 Maintenance

Lifetime of the chair is influenced by its use, storage, regular maintenance, servicing and cleaning.

### 4.1 Regular Maintenance

The maintenance activities to ensure a good condition for your Vermeiren chair are described below:

- Before every use
  - Check the wheels are clean.
  - Check the brakes are working.
  - Check screw fastenings are secured.
  - Check the chair (cleanliness, cracks, damage on structural parts ... .) and clean if necessary.
- Every 8 weeks: inspection, possibly lubrication and adjustment of
  - Tilt mechanism
  - Adjusting mechanism (seat height)
- Every 6 months or for every new user
  - General review
  - Cleanliness, Disinfection
  - Function of the wheels: If the rotation resistance is too large, clean the bearings of the steering wheels. If this is insufficient, please consult your specialist dealer.

For your convenience, on the back of this manual is a maintenance plan.

Repair and assembly of spare parts for your chair may only be performed by the specialist dealer.

Only authorized Vermeiren spare parts may be installed.

### 4.2 Shipping and storage

The shipping and storage of the product shall be according following instructions:

- Store in a dry place (between +5 °C and +41 °C).
- The relative humidity of the air should be between 30% and 70%.
- Provide sufficient covering or packaging to protect the product from rust and foreign bodies. (e.g. salt water, sea air, sand, dust).
- Store all removed parts together in one place (or mark them if necessary) to avoid mixing up with other products when re-assembling.
- Components must be stored without being subjected to strains (Do not put too heavy parts on the chair, not clamping between something, ...).

### 4.3 Care

#### 4.3.1 Upholstery

The cleaning of the upholstery shall be according:

- Clean upholstery with a cloth moistened with hot water. Be aware that you do not soak the upholstery.
- Use a mild commercial detergent for removing stubborn dirt.
- Stains can be removed by using a sponge or a soft brush.
- Do not use strong cleaning liquids like solvents, nor use hard brushes.
- Never clean with steam and/or pressure washers.

### 4.3.2 Plastic parts

Clean plastic parts of your chair with commercial plastic cleaners. Only use a soft brush or soft sponge. Please comply with special product information.

### 4.3.3 Coating

The high quality of the surface layer guarantees optimal protection against corrosion. If the outer coat is damaged by scratching or in some other way, get your specialist dealer to repair the affected surface.

When cleaning, only use warm water, normal household detergents, soft brushes and cloths. Ensure that no moisture gets into the tubes.

Initially the zinc parts only require rubbing with a dry cloth. Stubborn dirt can best be removed by using a suitable commercial zinc polish.

## 4.4 Inspection

In principle we recommend one inspection every year, and a minimum of one before usage is resumed. All of the following checks must be performed and documented by authorized persons:

- Check the frame parts for plastic deformation, cracks and impaired functioning.
- Visually check for damage to the paintwork (danger of corrosion).
- Check the operation of the wheels (free running, level rolling, axle play).
- Check the solidity and tightness of all screws, nuts and bolts.
- Verify the amount of grease on the metal joints of movable parts.
- Visually check all plastic parts for cracks and brittle spots.
- Check the function of the leg rest (locking, loading, deformation, wear and tear due to loading).
- Check the operation of other detachable parts (example: safety belt).
- Check the inclination system.
- Completeness of the delivery condition, instruction manual available?

The service must only be signed off in the maintenance plan if a minimum of all the above-mentioned aspects have been checked.

## 4.5 Disinfection

**⚠ WARNING: Dangerous Products - The use of disinfectants is restricted to authorized personnel.**

**⚠ WARNING: Dangerous Products, change on irritate your skin - You should wear suitable protective clothing because the disinfectants could irritate your skin. For this purpose you should also take note of the product information of the solutions concerned.**

All parts of the chair can be treated by scrubbing with a disinfectant.

All steps taken to disinfect rehabilitation equipment, their components or other accessory parts are to be recorded in a disinfection report containing a minimum of the following information (with product documentation appended):

Date of the disinfection	Reason	Specification	Substance and concentration	Signature
--------------------------	--------	---------------	-----------------------------	-----------

Table 3: Example of a disinfection book

**Abbreviations used in column 2 (reason):**

**V = Suspected infection IF = Infection case W = Repetition I = Inspection**

For a blank sheet of a disinfection book see § 9.

The recommended disinfectants for scrubbing (based on the list provided by the Robert Koch Institute, RKI) are listed in the table below. The current state of the disinfectants included in the RKI list can be obtained from the Robert Koch Institute (RKI) (homepage: [www.rki.de](http://www.rki.de)).

Active substance	Product name	Laundry disinfection		Surface disinfection (scrubbing-wiping/disinfection)		Disinfection of excretions 1 part sputum or stools + 2 parts diluted solution or 1 part urine + 1 part diluted solution						Area of effectiveness	Manufacturer or Supplier
		Diluted solution	Time to take effect	Diluted solution	Time to take effect	Sputum		Stools		Urine			
						Diluted solution	Time to take effect	Diluted solution	Time to take effect	Diluted solution	Time to take effect		
		%	Hr.	%	Hr.	%	Hr.	%	Hr.	%	Hr.		
Phenol or phenol derivative	Amocid	1	12	5	6	5	4	5	6	5	2	A	Lysoform
	Gevisol	0,5	12	5	4	5	4	5	6	5	2	A	Schülke & Mayr
	Helipur			6	4	6	4	6	6	6	2	A	B. Braun
	m-cresylic soap solution (DAB 6)	1	12	5	4							A	
	Phenol	1	12	3	2							A	
Chlorine, organic or inorganic substances with active chlorine	Chloramin-T DAB 9	1,5	12	2,5	2	5	4					A <sup>1</sup> B	
	Clorina	1, 5	12	2,5	2	5	4					A <sup>1</sup> B	Lysoform
	Trichlorol	2	12	3	2	6	4					A <sup>1</sup> B	Lysoform
Per combinations	Apesin AP100 <sup>2</sup>			4	4							AB	Tana PROFESSIONAL
	Dismozon pur <sup>2</sup>			4	1							AB	Bode Chemie
	Perform <sup>2</sup>			3	4							AB	Schülke & Mayr
	Wofesteril <sup>2</sup>			2	4							AB	Kesla Pharma
Formaldehyde and/or other aldehydes or derivatives	Aldasan 2000			4	4							AB	Lysoform
	Antifect FD 10			3	4							AB	Schülke & Mayr
	Antiseptica surface disinfection 7			3	6							AB	Antiseptica
	Apesin AP30			5	4							A	Tana PROFESSIONAL
	Bacillocid special			6	4							AB	Bode Chemie
	Buraton 10F			3	4							AB	Schülke & Mayr
	Desomed A 2000			3	6							AB	Desomed
	Hospital disinfectant cleaner			8	6							AB	Dreiturm
	Desomed Perfekt			7	4							AB*	Desomed
	Formaldehyde-solution (DAB 10), (formaline)	1,5	12	3	4							AB	
	Incidin Perfekt	1	12	3	4							AB	Ecolab
	Incidin Plus			8	6							A	Ecolab
	Kohrsolin	2	12	3	4							AB	Bode Chemie
	Lysoform	4	12	5	6							AB	Lysoform
	Lysoformin	3	12	5	6							AB	Lysoform
	Lysoformin 2000			4	6							AB	Lysoform
	Melsept	2	12	4	6							AB	B. Braun
	Melsitt	4	12	10	4							AB	B. Braun
	Minutil	2	12	6	4							AB	Ecolab
	Multidor			3	6							AB	Ecolab
Nüscosept			5	4							AB	Dr. Nüsken Chemie	
Optisept			7	4							AB*	Dr. Schumacher	
Pursept-FD			7	4							AB*	Merz	
Ultrasol F	3	12	5	4							AB	Fresenius Kabi	
Amphoteric surfactants (amfotensiden)	Tensodur 103	2	12									A	MFH Marienfelde
Lye	Lime-milk <sup>3</sup>							20	6			A <sup>3</sup> B	

1 Not effective against myco-bacteria when service disinfecting, especially in the presence of blood.  
2 Not suitable for disinfecting blood-contaminated or porous surfaces (e.g. raw wood).  
3 Useless for tuberculosis; preparation of Lime-milk: 1 part dissolved lime (calcium hydroxide) + 3 parts water.  
\* Checked for effectiveness on viruses in accordance with checking methods of the RKI (Federal Health Reporting 38 (1995) 242).  
A: Suitable for killing vegetative bacterial germs including myco-bacteria as well as fungi, including fungal spores.  
B: Suitable for deactivating viruses.

Table 4: Disinfectants

Kindly consult your specialist dealer if you have queries on matters related to disinfection; he will gladly assist you.

## 5 Guarantee

Excerpt from the "General Business Conditions":

(...)

5. The guarantee period for warranty claims is 24 months for the Elios frame.

(...)

The guarantee excludes damage arising from structural changes to our products, insufficient maintenance, defective or improper handling or storage or the use of non-original parts. Likewise, the guarantee excludes parts or working parts subject to natural wear and tear.

(...)

## 6 Disposal

When disposing of the product, contact your local disposal centre or return the product to your specialist dealer who, after submitting it to a hygienic procedure, will be able to send it back to the manufacturer who will dispose of and recycle it correctly, separating it into its component materials.

Packaging materials can be taken to disposal or recycling centers or to your specialist dealer.

## 7 Declaration of conformity

***The manufacturer or his authorized representative :***

N.V. VERMEIREN N.V

***Address :***

Vermeirenplein 1/15

2920 Kalmthout

Belgium

***declares under his sole responsibility that the CE marked medical devices :***

Productgroup: Shell chairs

Brand: Vermeiren

Type: Elios

***have been classified as class I, according to annex IX MDD 93/42/EEC, rule 1,***

***and is manufactured in full conformity with the European instructions below - including the latest modifications - and with the national law, that organizes this directions :***

Medical devices directive MDD 93/42/EEC: 2007

***and is in conformity with the relevant European harmonized standards:***

EN 12182: 1999, EN 12183: 2009

## 8 Maintenance plan

Date	Maintenance	Remarks	Paraph
1/1/2012	<i>Greasing and general servicing</i>	<i>non</i>	

## 9 Disinfection book

Date of the disinfection	Reason	Specification	Substance and concentration	Signature

Abbreviations used in column 2 (reason):  
V = Suspected infection    IF = Infection case    W = Repetition    I = Inspection

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## ENGLISH

### CONTRACTUAL WARRANTY

We offer 5 years of warranty on standard wheelchairs, lightweight wheelchairs 4 years. Electronic wheelchairs, tricycles, beds and other products: 2 years (batteries 6 months) and multiposition wheelchairs 3 years. This warranty is limited to the replacement of defective or spare parts.

### APPLICATION CONDITIONS

In order to claim this warranty, part "B" of this card has to be given to your official Vermeiren dealer. The warranty is only valid when parts are replaced by Vermeiren in Belgium.

### EXCEPTIONS

This warranty is not valid in case of:

- damage due to incorrect usage of the wheelchair,
- damage during transport,
- involvement in an accident,
- a dismount, modification or repair carried outside of our company and/or official Vermeiren dealership,
- normal wear of the wheelchair,
- non-return of the warranty card

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### Vermeiren Suisse S.A.
### Vermeiren Nederland B.V.